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Instruction manual Light weight wheel chair

Article No. KREHA-2063

The wheel chair REHASHOP „Bison 2 Blue“ serves to transport people in a sitting position. It can be used inside the house as well as for shopping purposes or on trips. The wheel chair is equipped with removable foot rests and a handbrake. This light weight wheel chair is also equipped with a drum brake for assistance as well as a rod for a fast dismantling of the back wheels. Due to the fact that the walking frame can be folded to save space, it is very suitable for car transports.

We wish you more mobility with your REHASHOP „Bison 2 Blue“ wheelchair.

- | | |
|---|------------------------------|
| 1 | Handles |
| 2 | Arm rest |
| 3 | Leg Backing |
| 4 | Grip screw |
| 5 | Back wheel |
| 6 | Handbrake |
| 7 | Steerable front wheel |
| 8 | Adjustable foot rest |

Wheel chair mounting

Make sure that all parts are complete before you start setting up your walking frame step by step:

- Remove the entire packaging material.
- Unfold the running gear as described and shown in the category „unfolding of the wheel chair“.
- Mount the foot rests as described and shown in the category „foot rests“.

Unfolding of the wheel chair

To unfold the wheel chair push the seat tubes down to the rest. This is easier done if you tilt the wheel chair a little bit to take weight off of the drive wheel.

Folding of the wheel chair

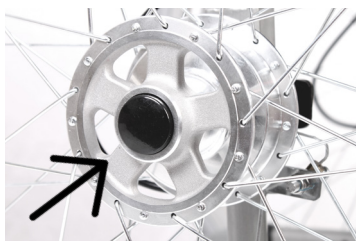
First of all make sure to remove any pillows and to fold up the footrests. To fold the chair, grab underneath the cover and pull it up. To fold the wheel chair as much as possible simply take off the footrests. To do so, push the unlocking lever and turn the footrests away to the sides. Afterwards pull the footrest out of the crossbar of the wheel chair.

The rods of the drive wheel

The drive wheels of the lightweight wheel chair REHASHOP „Bison 2 Blue“ series are equipped with a rod and are therefore removable without the help of any tools. To take off the wheels simply push the adjusting knob – which is placed on the outside of the drive wheel axis – and pull out the wheel at the same time. Mounting is the opposite, you only need to pay attention that the locking of the rods snaps in. Make sure that the adjusting knob goes back into its original position and also check after mounting that the drive wheels are in a firm position.

Handbrake

The wheel chair is equipped with two handbrakes. The brakes affect the drive wheels directly. To lock, simply push the lever all the way forward to the block. To unlock the handbrake, pull the lever all the way up to the back.

X = adjusting knob for rear axle (image similar)

The braking effect decreases if:

- the wheels are worn out
- there is too little air pressure (if you chose air filled wheels)
- the wheels are wet
- the brake is not adjusted properly

The handbrake is not a service brake! Do not use it while driving the wheel chair. Always stop the wheel chair with the help of the grip screws which are located on the drive wheels or with the help of the assistant brake. Make sure that the distance between the handbrake and the wheels is at least 3 mm. To adjust the brake simply loosen up the fastening screw of the brake which is located on the frame and adjust it to the required distance.

Attention! After the adjustment of the drive wheel, the brakes must be checked because they might need an adjustment as well.

Attention! Worn out wheels can cause an impairment of the brakes which is why the braking effect should be checked on a regular basis.

The clamp (Anti-tilt mechanism)

The clamp can be used to tilt over the wheel chair from behind through a second person. If the assistant steps on the clamp the wheel chair can be easily pushed over the curb.

Side parts and arm rests

The side parts are equipped with a plastic filling as well as a padded arm rest. The side parts can be removed if needed (for example to get closer to a table). To remove the side parts simply push both unlocking knobs at the same time and pull up the side part. The mounting takes place in the opposite

order. Please pay attention while mounting that the unlocking snaps in correctly. Important: Side parts are not meant for lifting up or carrying the wheelchair.

The foot rests

The wheel chair is equipped with footrests. These footrests can be fold away for an easy access and exit. The footrest can be individually adjusted to the users needs by pulling out the pins along the steel tube, moving the footrest into the desired position and setting them back in place. Please note that there must be a minimum space of 2,5 cm between the footrests and the ground.

The foot rests can be turned to the inside and outside. Place the foot rests to the outside for insertion then insert them and push them softly to the inside until the locking snaps in. To remove the foot rests just push the unlocking knob and turn the foot rests to the outside. When mounting the foot rests always make sure that they are snapped into place correctly. Important: The foot rests are not meant to carry or lift up the wheel chair with somebody sitting in it.

Wheels and wheel mounting

The light weight wheelchair REHASHOP „Bison 2 Blue“ will be delivered with antibreakdown solid rubber wheels. Therefore no air pressure check is needed. Worn out or old wheels should be replaced by a specialist dealer.

Cleaning and maintenance

To clean the wheel chair simply use water and if wished a mild cleanser. You can remove tough dirt by using a car cleanser. No water should enter the wheel chair tubes during the cleaning process which is why it should only be cleaned in a fully mounted position. If needed, have your wheel chair checked and maintained by a professional or replace a worn out one with a new one. Make sure to contact your specialist dealer when you find any defects while checking the braking force on a regular basis. It is not permitted to operate the wheelchair without professional maintenance.

GB

Do the following maintenance work on a regular basis:

- Check the wheels for wear out every four weeks.
- Check the tightness of the wheelchairs bolted connections every six months.
- Lubricate the rods every eight weeks with sewing machine oil.
- Seawater, de-icing salt and sand damage the bearing and the steering gear of the wheel chair. After the wheelchair comes in touch with the mentioned things make sure to clean it with fresh water.

Common security advice

Only use the wheel chair if it is in flawless condition. Improper use of the wheel chair can put the user in danger. For your own safety we strongly suggest you to follow the following regulations:

- Unprofessional or missing adjustments can increase the risk of accidents.
- We would like to point out to you that you are part of the public road traffic as a wheelchair user and therefore you must follow the road traffic act.
- Wear light colored clothes in darkness.
- When driving the first time be very careful to adjust to the wheelchairs shift of emphasis impact.
- Exercise caution when driving on even grounds, inclines and declines.
- The handbrake is not meant to slow down the wheelchair while driving!!! It is only meant for unwanted rolling away while standing. If you stop on uneven ground make sure to use both handbrakes. The wheel chair can tilt over if you only use one of the handbrakes.
- If available use ramps and elevators. In case there

are no ramps or elevators on your way you should take an assistant with you to reach your destination. The assistant may only hold onto the wheelchairs frame parts. It is not allowed to lift up the wheelchair with somebody sitting in it - make sure to roll it.

- Before you start always sure to check the rods on the drive wheels, the velcro of the seat and backrest, the wheel possibly the air pressure of the wheels as well as the brakes.
- The maximum load weight is 125 kg.
- The wheel chair serves exclusively as transportation of people sitting on the seat. Another use is considered to be improper.
- It is not allowed to use the wheelchair while riding a bus or train! Only the seats of the bus or train give enough protection in dangerous situations.
- It is important to read the right part of the instruction manual if you wish to make any changes to the wheelchairs adjustments.
- Especially when driving up inclines or driving down declines, road holes or uneven areas which can cause the wheel chair to tilt over. It is advisable to lean forward if going over a step or while driving up an incline.
- It is only recommended to use the wheel chair on solid ground.
- The wheel chair is not meant as support to lift yourself up from a squatting or sitting position.
- The wheels of the wheelchair can get stuck if used on uneven grounds like grooves or grids. Try to avoid this kind of obstacle.
- Activate the handbrake if parking the wheelchair. This avoids unintended rolling away of the wheelchair.
- Make sure to only put weight on the seat from above!
- Worn out wheels can affect the braking force of the wheelchair. This is why you should make sure that the braking force of the brakes is in a good working condition.
- Slip hazard on wet, dirty, iced or snow covered ground! This causes a severely limited braking force. Adjust your handling to these dangers.
- Even if all moveable parts have been removed the wheel chair still has a weight of 10 kg. This is why a lift should only be done in a folded condition by grabbing the handles and front frame.

- If problems or questions occur please contact your specialist dealer.

Warranty

We grant warranty on this product according to European provision of the law.

Warranty is valid for the first 12 months. This warranty is valid for production and mounting defects. We are not liable for wear and tear and careless or improper use. The warranty loses its validation if changes or reparations have been made by non authorized dealers or people. This does not include costs for troubleshooting, freight, usage, loss of earnings, needed drives and similar costs. Also not included in the warranty are damages and injuries which were caused by improper use, abuse or disregard of the instruction manual. It is forbidden to make any technical changes to this product.

Technical data

Deliverable seat width: 42 cm, 45 cm, 51 cm

Seat depth: approx. 41 cm

Wheelchair weight (depending on the model): approx. 15,0 kg

Back wheels: 24" approx. 61 cm Ø

Front wheels: 7" approx. 18 cm Ø

Arm rests: removable and turnable

Leg rests: removable and height adjustable

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KAPHINGST ONLINE AG
Proteno GmbH