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Instruction manual REHASHOP bed table Robeti

Articelno. KREHA-1006

1. Rack
2. Table top
3. Hand wheel (tiltable)
4. Locking pin (height adjustment)
5. Hand wheel (height adjustment)
6. Crossbar
7. Handbrake
8. Running gear
9. Hand wheels fastening
10. Rollers

GB

The product

The „REHASHOP bed table Robeti“ serves as a tray or rack in combination with a bed for elderly people or a hospital bed.

It is the perfect support to eat meals in bed or to use it as book support. The bed table is height adjustable and the table top can be easily tilted to individual needs. The table is equipped with rollers and can be easily moved around. Due to its special solid construction it can be placed with its base part under the hospital bed.

The table top of the „REHASHOP bed table Robeti“ has a nice beech tree design with a synthetic coating. The surrounding edges are bordered with a solid beech tree wood frame.

The mounting of the bed table

The REHASHOP bedtable Robeti consists of three parts which have to be assembled. Please check if all

parts shown on the picture have been delivered and mount the bedtable in the following sequence:

1. Remove all the packing material
2. Start to mount the bedtable with lower part of the frame. Mount the 4 wheels by putting them into the frame. Mount the lower part with two handwheels on the bottom of the frame (No. 9). Check if the safety pin (No. 4) is mounted in the correct position (See picture)
3. Assemble the tabletop and put the upper part of the frame into the tube of the lower part of the frame. Check if the safety pin fits into one of the holes. The shorter screw will be needed to lock the height adjustment (No. 5). The larger screw is needed for the adjustment of the angle of the table top (No. 3).
4. Loosen the larger screw and flip the table top in a horizontal position. Lock the table top by fastening this screw. Make sure that the screw fits into one of the deepenings.

5. Mount the small table top with the remaining handwheel to the frame.

Height adjustment

The height of the table top can be adjusted with the help of the hand wheel and the locking pin.

GB

Loosen the hand wheel of the frame and hold the crossbar with your hand while you pull the locking pin outwards. After you loosened the locking pin you can move the crossbar up or down.

Important: Make sure that the locking pin snaps in before you tighten the hand wheel again.

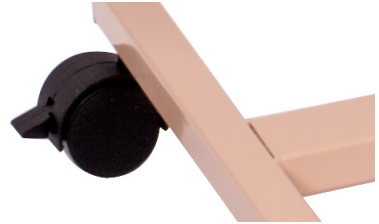
Crossbar with hand wheel and locking pin.

Handbrake

The „REHASHOP bed table Robeti“ is equipped with two handbrakes on two of the wheels. To activate and to lock the brakes use your foot to push down both plastic levers. To unlock the handbrake pull up both locking levers.

Important: Secure the „REHASHOP bed table Robeti“ with the handbrakes against accidental rolling away. Especially on slippery and uneven grounds the danger of rolling away is present.

Handbrake

The rack

The rack in beech tree design is meant for the storing of little things. It is not allowed to use the rack as seat. Important: To avoid damages do not put hot, heavy or sharp things on the rack.

Cleaning and maintenance

To clean the bed table simply use water and if you wish a mild cleanser. You can remove tough dirt by using a car cleanser. No water should enter the bed table tubes during the cleaning process which is why it should only be cleaned in a fully mounted position. If needed, have your bed table checked and maintained by a professional or replace a worn out one with a new one. Make sure to contact your specialist dealer when you find any defects while checking the braking force on a regular basis. It is not allowed to operate the bed table without professional maintenance.

Common security advices

- Operate the „REHASHOP bed table Robeti“ only if it is in a flawless condition.
- It is only recommended to operate the bed table on an even, solid ground.
- Do not use the bed table as support to get up from a squatting or sitting position also do not use it to transport people with it.
- The wheels of the bed table can get stuck if used on uneven grounds like grooves or grids. Try to avoid this kind of obstacle.
- Activate the handbrake if „parking“ the bed table. This avoids unintended rolling away of the bedtable.
- Worn out wheels can affect the braking force of the handbrake. That is why you should make sure that the braking force of the brakes is in a good working condition.
- If problems or questions occur please contact your specialist dealer.

Technical data

Total width:	400 mm
Total length:	800 mm
Height (adjustable):	760 - 1080 mm
Shelf:	200 x 400 mm
Table top (adjustable):	590 x 400 mm
Total weight:	10,0 kg

Rollers:	Rollers made of plastic, diameter 50 mm, 2 rollers are fixable.
----------	---

Warranty

We grant warranty on this product according to European provision of the law. Warranty is valid for the first 12 months. This warranty is valid for production and mounting defects. We are not liable for wear and tear and careless or improper use. The warranty loses its validation if changes or reparations have been made by non-authorized dealers or people. This does not include costs for troubleshooting, freight, usage, loss of earnings, needed drives and similar costs. Also not included in the warranty are damages and injuries which were caused by improper use, abuse or disregard of the intrusion manual. It is forbidden to make any technical changes to this product.

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KAPHINGST ONLINE AG
Proteno GmbH