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**D**

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**NL**

Instruction manual walking frame

**GB**

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**F**

**REHASHOP Vita**

Art. No.: KREHA-1001

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Artikelnr.: KREHA-1001

D

## Das Produkt

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**GB**

## The product

The walking frame supports the user during walking. It can be either used inside the house, for shopping purposes or on trips.

A major plus factor is the removable basket as well as the removable tray. There is also a big seat for frequent rests. Due to the fact that the walking frame can be folded to save space, it is very suitable for car transports.

1. Seal
2. Handle and brake
3. Protection against accidental folding
4. Cross bracing
5. Fixed back wheel
6. Steerable front wheel
7. Luggage frame
8. Locking lever
9. Tray
10. Basket

## Mounting of the walking frame

Make sure that all parts are complete before you start setting up your walking frame step by step:

1. Remove the entire packaging material as well as the cardboard between the back wheels.
2. Unfold the running gear as described and shown in the category „folding and unfolding of the walking frame“.
3. Place the walking frame in front of you with the front wheels in the forward position.
4. Slide the handlebar rod into the crossbar of the running gear. Make sure that the handlebar rod has been placed in there at least 10 cm deep.

5. Set the handlebar rod clockwise on both sides with the locking lever. Make sure to adjust it to the right length the user needs.
6. Bring the end of the locking lever in a downwards facing position. To do so just pull the locking lever outwards and then

turn it downwards. The effect of the locking lever does not change through this.

7. Make sure that the protection against accidental folding (colored red) on the running gear is pushed down and snapped into place.
8. Attach the basket to the running gear and place the tray on the seat. Please pay attention to place the trays sticks into the holes of the seat.

## Folding and unfolding of the walking frame

To unfold just pull the back wheels backwards. Now push down the cross bracing and push down the red colored protector against accidental folding until it snaps in. This integrated security feature protects the walking frame from accidental folding during the usage of it. The usage of the walking frame without this security feature is strongly prohibited! Disregarding can cause serious injuries! Especially if you overcome an obstacle without the protector snapped into place, the walking frame can collapse and cause several injuries!

Before folding up the walking frame, make sure to remove the basket as well as the tray. Fold up the red colored protection against accidental folding and then pull up the cross bracing in an upright position to finally fold up the walking frame.

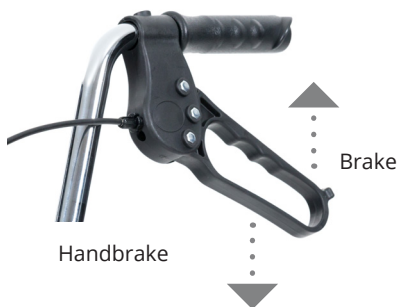
## The locking lever

With the help of the locking lever, the height of the ergonomically shaped handles can be easily and individually adjusted to the user. By turning the locking lever counterclockwise the handles become loose and you can pull them out or put them deeper into the frame. Please make sure that at least 10 cm of the handlebar remains inside the crossbar of the walking frame. This is assured if no plain metal part is visible below the riffled area of the

handlebar. Secure the handlebar again by turning the locking lever clockwise. After the handlebar has been tightened, the locking lever must be in a downward facing position again. In case it is not, please pull the lever outwards and bring the locking lever in a downward facing position. The effect of the locking lever does not change through this.

## Brake function of the walking frame

To slow down the walking frame, simply pull both brake handles towards you. To fix the handbrake, simply push the brake handles in the opposite direction until you hear a clicking sound. To unlock the handbrake pull up both brake handles until the brake unlocks again.

**Attention:** Worn out plastic wheels can influence the effectiveness of the brakes. That is why the brakes should be controlled regularly.

## The basket

The basket makes your shopping truly comfortable. It can be attached via hooks below the seat.

**GB**

Attention: The basket is only suitable for the transport of groceries and similar things. It is not suitable to place children or animals in. The maximum loading weight is 5 kg. It is highly recommended not to overload the basket since this can cause accidents and damage the basket.

## The seat

The seat is made of plastic. It allows the user to have brakes while walking and to take the burden off of the legs. Attention: Do not use the tray as a seat and make sure that the handbrake on the left and the right side have been fully pressed down. You can find a detailed description of how to use the handbrake in the category „brake function of the walking frame“.

Make sure that the only pressure on the seat is from above. Do not lean against the seat or the walking frame because it is no substitute for a chair. The walking frame requires an even, solid ground in order to use the seat.

## The cane holder

The walking frame is equipped with a cane holder. The cane holder is placed on the right side and consists of a clip, which is on the same level as the seat, as well as plastic box, which is on the same level as the luggage rest. If using the cane holder make sure that the cane is fixed and secured.

## Cleaning and maintenance

To clean the walking frame, simply use water and if you wish a mild cleanser. You can remove tough dirt by using a car cleanser. No water should enter the walking frame tubes during the cleaning process which is why it should only be cleaned in a fully mounted position. If needed, have your walking frame

checked and maintained by a professional or replace a worn out one with a new one. Make sure to contact your specialist dealer, when you find any defects while checking the braking force on a regular basis. It is not allowed to operate the walking frame without professional maintenance.

## Warranty

We grant warranty on this product according to european provision of the law. Warranty is valid for the first 12 month. This warranty is valid for production and mounting defects. We are not liable for wear and tear as well as careless or improper use. The warranty loses its validation if changes or reparations were done by non authorized dealers or people. It does not include costs for troubleshooting, freight, usage, loss of earnings, needed drives and similar costs. Furthermore the warranty does not include damages and injuries which were caused by improper use, abuse or disregard of the instruction manual. It is forbidden to make any technical changes to this product.

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Total width:	66,5 cm
Total length:	60 cm
Width between frontwheels:	38 cm
Seat height:	61 cm
Height of the handlebars:	79 - 94 cm
Width between the handlebars:	40 cm
Total weight:	10,5 kg
Diameter wheels:	20 cm
Max. weight loading:	130 kg
Max. weight loading tray:	5 kg
Max. weight loading basket:	5 kg
Max. weight loading luggage rest:	10 kg

## Common security advices

- Operate the walking frame only if it is in a flawless condition.
- Before every use, check if the locking lever is tightened and the handlebar is in a downwards facing position.
- Operate the walking frame only with a snapped in protection.
- It is only recommended to operate the walking frame on an even, solid ground.
- Do not use the walking frame as support to get up from a squatting or sitting position also do not use it to transport people with it.
- The wheels of the walking frame can get stuck if used on uneven grounds like grooves or grids. Try to avoid this kind of obstacles.
- Always hold on to the handlebars with both hands. If you only use one hand there is always the danger of losing balance and falling over.
- Always use the brakes of the walking frame slowly and evenly.
- Activate the handbrake if parking the walking frame. This avoids an unintended rolling away of the walking frame.
- Make sure to only put weight on the seat from above!
- The maximum weight load of the basket is 5 kg.
- Worn out wheels can affect the braking force of the walking frame. That is why you should make sure that the braking force of the brakes is in a good working condition.
- Slip hazard on wet, dirty, iced or snow covered ground! This causes a severely limited braking force. Adjust your handling to these dangers.
- If problems or questions occur, please contact your specialist dealer.

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PROTENNO GmbH
Kaphingst Online AG