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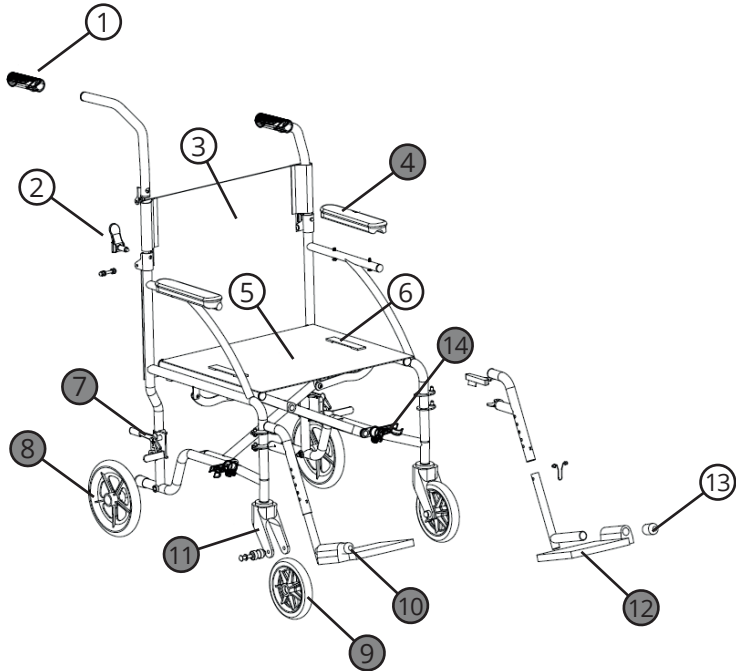
[www.verpleeg-shop.nl](http://www.verpleeg-shop.nl)

PROTENOM GmbH  
Stationsplein 8k  
6221 BT Maastricht  
NEDERLAND

## Instruction manual Transport Chair Antilope 3

Art. No. REHA-2006


No.	Spare part	Art. No.
4	Armrests for Antilope 3	REHAET-0175
7	Wheel lock for Antilope 3 (1 pair)	REHAET-0176
8	Rear wheel for Antilope 3	REHAET-0177
9	Front wheel for Antilope 3	REHAET-0178
10	Footrest left for Antilope 3	REHAET-0180
11	Caster forks for Antilope 3	REHAET-0179
12	Footrest right for Antilope 3	REHAET-0181
14	Footrestclip for Antilope 3 (1 pair)	REHAET-0182

## Instruction manual Transport Chair Antilope 3

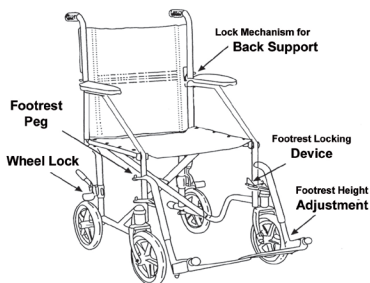Art. No. REHA-2006

It is important for the attendant to be familiar with proper assisting procedures in any situation that requires the use of the transport chair. Safety is the key consideration. Consult your physician, nurse or physical therapist to determine methods most suitable for your individual needs and abilities.

Note: The transport chair does not conform to federal standards for automobile seating and should not be used as a seat in a motor vehicle.

### Operating tips: How to operate your transport chair

1. To open the chair, tilt to one side and push down on seat rails.

2. Lift back support to upright position. The lock to secure the back will engage automatically. To fold down and store, pull up on locks simultaneously and carefully fold down the back support.

GB

3. To attach footrests, align the holes on the footrest arm with the pegs on the frame of the transport chair.

4. To adjust the length of the footrests, loosen the bolt (or knob) on the telescoping tube shaft. Tighten securely after adjustment. The lowest part of the footrest should not be less than 2,5 inches from the ground to permit proper clearance.

5. To remove the swing-away footrest, release the lock. Swing outward and rearward. Simply lift up and off to remove the footrest.

6. The safety belt is already attached for use and should not be removed from the transport chair.

Note: The transport chair should not be operated without the assistance of an attendant.

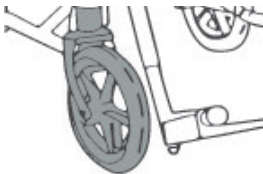
### Safety tips:

Please read carefully before using your transport chair

1. Before entering or leaving the chair, engage the dual wheel locks against the tires on both rear wheels. Make sure that the front casters are in the forward position before transferring into or out of the chair.

2. Do not slide forward on the seat while leaning ahead. Otherwise there is an increased risk of falling.

If you want to pick up something from the ground, drive your wheelchair behind the object and move back.

By moving backwards the front wheels are automatically aligned forwards and brought into the correct position.

3. Leaning out of the transport chair without proper assistance could cause tipping.

4. Do not step on the floor plates when transferring, as this could cause tipping.

Note: You and your attendant should consult a qualified professional for correct techniques on ramp negotiations.

### Maintenance tips: maintaining your transport chair

Proper care and maintenance is important in order to keep your transport chair in good working condition. Be sure that your chair is in proper working condition all the times. A qualified dealer or repair center (wheelchair or rental dealer) should be consulted for periodic inspections and repairs.

1. At least once a week, wipe off chrome using a clean soft cloth. If the chair is exposed to moisture, dry the chair immediately. The metal parts of the chair should be polished one a month with car wax.

2. Clean the upholstery with a mild soap solution, rinse and wipe dry. Worn or torn upholstery should be replaced promptly as it may not support your body weight.

3. Inspect the wheels at least once a month. Correct any side play by loosening the lock nut on the axle, then tighten the axle just enough to remove side play.

4. Solid rubber tires require limited upkeep, but need to be cleaned periodically with a damp cloth. Solid rubber tires need to be replaced if they become cracked or severely worn.

## Warranty

We grant warranty on this product according to European provision of the law.

Warranty is valid for the first 12 months.

This warranty is valid for production and mounting defects. We are not liable for wear and tear and careless or improper use. The warranty loses its validation if changes or reparations have been made by non authorized dealers or people. This does not include costs for troubleshooting, freight, usage, loss of earnings, needed drives and similar costs. Also not included in the warranty are damages and injuries which were caused by improper use, abuse or disregard of the instruction manual. It is forbidden to make any technical changes to this product.

## Technical data

Deliverable seat width:	47 cm
Seat depth:	approx. 40 cm
Folded (depending on the model):	ca. 23 cm
Wheelchair weight (depending on the model):	approx. 14,5 kg
Wheels:	approx. 23 cm Ø
Leg rests:	removable and height adjustable
Seat padding:	black nylon
Frame:	Steel
Maximum user weight:	135 kg

The turning moment for the drive wheels fastening screws is 10 Nm.

The turning moment for all other wheel chairs screws is 8 Nm.

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PROTENO GmbH
Kaphingst Online AG