**AUSTRIA**

Bauerfeind Ges.m.b.H.  
Hainburger Straße 33  
1030 Wien  
P +43 (0) 800 4430-130  
F +43 (0) 800 4430-131  
E info@bauerfeind.at

**BENELUX**

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en english

**These instructions for use can also be downloaded from:**  
[www.bauerfeind.com/downloads](http://www.bauerfeind.com/downloads)

### Dear Customer,

thank you for choosing a product from Bauerfeind.

If "custom-made solution" is printed on the sewn-in label and there is no CE mark, this means it is a made-to-measure product specially manufactured in accordance with Section 3 No. 8 of the German Medical Devices Act.

Please read the **instructions for use** carefully and contact your physician or medical retailer if you have any questions.

### Intended use

The MalleoTrain ankle support is particularly effective during movement and therapeutic exercises. The elasticated knitted fabric and integrated anatomically shaped pads made from a viscoelastic material compress and massage the soft tissue parts of the joint.

Enhanced circulation in the joint and promotion of lymph drainage lead to reduced swelling, pain relief and improved function.

### Indications

- Post-operative and post-traumatic irritation (e.g. after sprains)
- Joint effusion and swelling associated with degenerative changes in the ankle (osteoarthritis) and joint inflammation (arthritis)
- Muscle and tendon disorders (tendomyopathies)
- Ligamental weaknesses

### ⚠ Risks of using this product

- The use and duration of use must be discussed with the physician who prescribed the MalleoTrain. They will also inform you whether the MalleoTrain can be used simultaneously with other products, and if so, which ones.
- Please note all the information you received with the MalleoTrain as well as the instructions for use.
- The product may be damaged if it is not used correctly or is altered arbitrarily and the warranty will not be valid.
- The effects of this support are felt primarily during physical activity. Take the support off during extended rest phases – e.g. sleeping and long periods of sitting down.
- Avoid contact with ointments, lotions, or substances containing grease or acids.
- Any aids applied externally to the body may, if tightened excessively, lead to local pressure points or, rarely, constrict the underlying blood vessels or nerves.
- If you experience any unusual changes connected to the use of the MalleoTrain – e.g. an increase in discomfort – stop using it and contact your physician without delay.

 Contains natural rubber latex (Elastodiene) which can cause skin irritation in persons with an allergy. Take off the support if you notice any allergic reaction.

### Contra-indications

No side effects that affect the entire body have been reported to date.

If you have any of the following conditions, the product should only be used after consultation with your physician:

1. Skin disorders/injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness, and excessive heat build-up.
2. Impaired sensation and circulatory disorders of the legs/feet (e.g. diabetes mellitus)
3. Impaired lymph drainage – including soft tissue swellings of uncertain origin located away from the fitted aid.

### Fitting MalleoTrain

1. Fold the upper part of the support outwards until you can grip the heel. ❶
2. Pull the support over the foot up to the heel. ❷
3. Fold back the folded-over part of the support.
4. Then pull it up until the bones of the ankle rest in the rounded hollows of the pads. ❸
5. Adjust the fit of the support as necessary. ❹

### Product Care Instructions

**Note:** Never expose the support to direct heat, e.g. heater, sunlight, leaving in your car, etc. This can impair the effectiveness of the support.

Please refer to the care instructions on the sewn-in label of your MalleoTrain. This can be found on the upper edge of the support.

Please note the following:

1. We recommend using a lingerie bag.
2. The support is not suitable for tumble drying.

Wash your support regularly to retain the compression of the knitted fabric.

To make a complaint, please contact the retailer that provided you with the product.

**Note:** Only clean goods are handled.

### Notes on reuse

This product is intended for your treatment only. Do not pass the support on to other people.

### Warranty

The statutory warranty regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. It is your first local point of contact before the manufacturer. Please use the MalleoTrain in line with expert instructions and the enclosed instructions for use. Take note of the cleaning instructions and do not make any alterations to the product. Only aids that function perfectly will help you. Failure to follow the instructions may also compromise the warranty.

### Disposal

Once you have finished using the product, please dispose of it according to local specifications.

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