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**GB**

## The product

The walking frame supports the user during walking. It can be either used inside the house, for shopping purposes or on trips. The walking frame is equipped with a removable basket. It is also provided with a wide padded seat for resting between the walking.

Due to the fact that the walking frame can be folded to save space, it is very suitable for car transport. Its frame is made of aluminium which makes it very lightweight and therefore easier to handle compared to heavy steel walking frames. We hope the REHASHOP light-weightwalking frame Vigeo supports you preserving your mobility.

1. Handles and brake with locking lever
2. Back rest
3. Locking knob
4. Padded seat
5. Removable Basket
6. Folding mechanism
7. Fixed back wheel

8. Steerable front wheel
9. Cane holder

## Mounting of the walking frame

Make sure that all parts are complete before you start setting up your walking frame step by step:

1. Remove the entire packaging material.
2. Unfold the running gear as described and shown in the category „Unfolding the walking frame“.
3. Place the walking frame in front of you with the front wheels in the forward position.
4. Slide the handlebar rod into the crossbar of the running gear. Make sure that the handlebar rod has been placed in there at least 15 cm deep.

- Holes for individual height adjustments
- Locking knob

5. Set the handlebar rod clockwise on the left and the right side by using both locking knobs. Make sure to adjust it to the right length the user needs. Always use the given holes on the handlebar rod.
6. Attach the basket to the running gear.

## Unfolding the walking frame

Place the walking frame in front of you with the front wheels in the forward position. Activate the handbrake. To unfold pull the back wheels backwards then use your hand to push down the seat until the folding mechanism snaps in. This integrated security feature protects the walking frame from accidental folding during the usage of it. To use the walking frame without an adjusted seat is prohibited and can cause serious injuries. Especially if you want to overcome an obstacle without the protector snapped into place, the walking frame can collapse and cause several injuries!

## Folding of the walking frame

Before folding up the walking frame, make sure to remove the basket. Place the walking frame in front of you with the front wheels in the forward position. Then activate the handbrake and finally grab the seat and pull it up. Be careful not to pin any clothes or body parts during the folding procedure.

## Locking knobs

With the help of the locking knobs, the height of the ergonomically shaped handles can be easily and individually adjusted to the user. By turning the locking knobs counter clockwise they become loose and can be fully removed. The handles can be pulled out more or can be inserted further into the frame. Make sure that at least 15 cm of the handlebar rod remains inside the walking frames crossbar.

This is assured if the given holes inside the tubes are used to fasten the handlebar. To fix the handlebar rod again turn the locking knobs clockwise. The handlebar rods cannot be over wound. The distance is eightfold adjustable each with 2,5 cm.

Pay attention to the approved fastening of the locking knobs while adjusting.

## The braking function of the

## walking frame

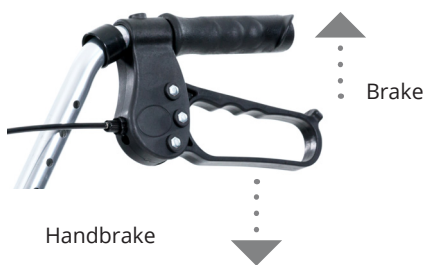
To slow down the walking frame pull both brake handles slowly up to the handles.

To activate the handbrake push the broke handle down in opposite direction until there is a snapping sound.

To unlock the handbrake, pull up both broke handles until the broke unlocks.

Important: Worn out plastic wheels can influence the effectiveness of the brakes.

This is why the brakes should be controlled regularly.

## The basket

The basket makes your shopping truly comfortable. It can be attached via hooks below the seat.

Attention: The basket is only suitable for the transport of groceries and similar things. It is not suitable to place children or animals inside. The maximum loading weight is 5 kg. It is highly recommended not to overload the basket since this can cause accidents and damage the basket.

## The seat

The wide and padded seat allows the user to have breaks while walking and to take the burden off of the legs. Important: Make sure that the handbrake on the right and the left side have been fully pressed down. The activation of the handbrake is described in detail in the category „braking function of the walking frame“. Make sure that the only

GB

pressure on the seat is from above and do not lean against the seat or the walking frame because it is not a substitute for a chair. The walking frame requires an even, solid ground in order to use the seat.

**GB**

## The cane holder

The walking frame is equipped with a cane holder. The cane holder is placed on the right side and consists of a clip, which is on the same level as the seat as well as plastic box, which is on the same level as the luggage rest. If using the cane holder make sure that the cane is fixed and secured.

## Cleaning and maintenance

To clean the walking frame simply use water and if you wish a mild cleanser. You can remove tough dirt by using a car cleanser. No water should enter the walking frame tubes during the cleaning process which is why it should only be cleaned in a fully mounted position. If needed, have your walking frame checked and maintained by a professional or replace a worn out one with a new one. Make sure to contact your specialist dealer when you find any defects while checking the braking force on a regular basis. It is not allowed to operate the walking frame without professional maintenance.

## Warranty

We grant warranty on this product according to European provision of the law. Warranty is valid for the first 12 months. This warranty is valid for production and mounting defects. We are not liable for wear and tear as well as careless or improper use. The warranty loses its validation if changes or reparations were done by non authorized dealers or people. It does not include costs for troubleshooting, freight, usage, loss of earnings, needed drives and similar costs. Furthermore the warranty is not including damages and injuries which were caused by improper use, abuse or disregard of the

instruction manual. It is forbidden to make any technical changes to this product.

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Total width:	60 cm
Total length:	72 cm
Width between frontwheels:	58 cm
Seat height:	55 cm
Height of the handlebars:	81 - 94 cm
Width between the handlebars:	43 cm
Total weight:	8 kg
Diameter wheels:	18,5 cm
Max. weight loading:	120 kg
Max. weight loading basket:	10 kg

## Common security advices

- Operate the walking frame only if it is in a flawless condition.
- Before every use, check if the locking lever is tightened and the handlebar is in a downwards facing position.
- Operate the walking frame only with a snapped in protection.
- It is only recommended to operate the walking frame on an even, solid ground.
- Do not use the walking frame as support to get up from a squatting or sitting position also do not use it to transport people with it.
- The wheels of the walking frame can get stuck if used on uneven grounds like grooves or grids. Try to avoid this kind of obstacles.
- Always hold on to the handlebars with both hands. If you only use one hand there is always the danger of losing balance and falling over.
- Always use the brakes of the walking frame slowly and evenly.
- Activate the handbrake if parking the walking frame. This avoids an unintended rolling away of the walking frame.
- Make sure to only put weight on the

seatfrom above!

- The maximum weight load of the basket is 10 kg.
- Worn out wheels can affect the braking force of the walking frame. That is why you should make sure that the braking force of the brakes is in a good working condition.
- Slip hazard on wet, dirty, iced or snow covered ground! This causes a severely limited braking force. Adjust your handling to these dangers.
- If problems or questions occur, please contact your specialist dealer.

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PROTENNO GmbH
Kaphingst Online AG