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Leichtgewichtrollator Butterfly, Transportrollator Campolina

D

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Lichtgewicht rollator Butterfly, Transportrollator Campolina

NL

## Instruction manual

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Kaphingst Online GmbH  
Stationsplein 8k  
6221 BT Maastricht  
NEDERLAND

# Instruction manual

Light-weight walking frame Butterfly Article No. KREHA-BUTTERFLY

Transport walking frame Campolina Article No. HCT-9103

GB

## INTRODUCTION

The rollator is manufactured from high quality materials and includes seat, bag, locking brakes and footrest for BRO199. The handles are adjustable in height and the rollator folds easily for transport or storage.

## INTENDED USE

Designed to assist the user to walk, rest and carry small items of shopping for indoor and outdoor use on flat surfaces by a single individual limited mobility with a maximum weight as shown below.

The intended user should possess the necessary physical, visual, and cognitive skills to operate the rollator safely. If you have any doubt, please consult your dealer/prescriber.

It is important that the user reads and understands the safety advice given within this manual before attempting to operate the product to avoid possible injury to the user or bystanders and avoid damage to the rollator or surrounding property.

## DO:

1. Always ensure the brakes are in the locked position if the rollator is left unattended.
2. Check all castors and wheels before using.
3. Check the brakes before using.
4. Check handle height adjustment hand knobs before using.
5. Check if the rollator is fully unfolded.
6. Take care while carrying the rollator folded.
7. Keep your hands away from the folding mechanism to prevent any risk or pinching.

## DO NOT:

1. Use the rollator as an aid to transport persons, it is not designed as a complete wheelchair.
2. Sit on the rollator if the brakes are not locked, this could cause the rollator to move resulting in injury to the user or other persons.
3. "Drop" into the seat of the rollator, always sit into the rollator in a smooth, controlled motion. Seek assistance if necessary.
4. Hang objects on the brake levers or hand grips. This could affect brake operation and/or instability of the rollator

Every care has been taken to ensure that your rollator will give you safety, reliable service provided you adhere to the advice given above.

## PRODUCT DIMENSIONS AND SPECIFICATION

### Light-weight walking frame Butterfly Article No. KREHA-BUTTERFLY

Overall Height	Overall Length	Overall Width	Seat Depth	Seating Width	Seat Height	Folded width	Castor Size	Unit Weight	Capacity
M in 79.5cm M ax 92.5cm	66cm	62cm	25cm	47cm	53cm	27cm	20 cm	7.6kg	130kg

GB

### Transport walking frame Campolina Article No. HCT-9103

Overall Height	Overall Length	Overall Width	Seat Depth	Seating Width	Seat Height	Folded width	Castor Size	Unit Weight	Capacity
Min 86cm Max 98cm	96cm	62cm	31cm	47cm	56cm	28cm	20cm	9.3kg	130kg

### Light-weight walking frame Butterfly Article No. KREHA-BUTTERFLY

1. Backrest
2. Adjustable parking / running brake
3. Seat area
4. Height adjustable handles
5. 200mm swivel front castors
6. Shopping bag
7. Cane holder
8. Rear leg folding knob

### Transport walking frame Campolina Article No. HCT-9103

1. Swivelling backrest
2. Adjustable parking / running brake
3. Seat area
4. Height adjustable handles
5. Detachable footrest
6. Shopping bag
7. Cane holder
8. 200mm swivel front castors

## ASSEMBLY INSTRUCTION

Light-weight walking frame Butterfly Article No. KREHA-BUTTERFLY

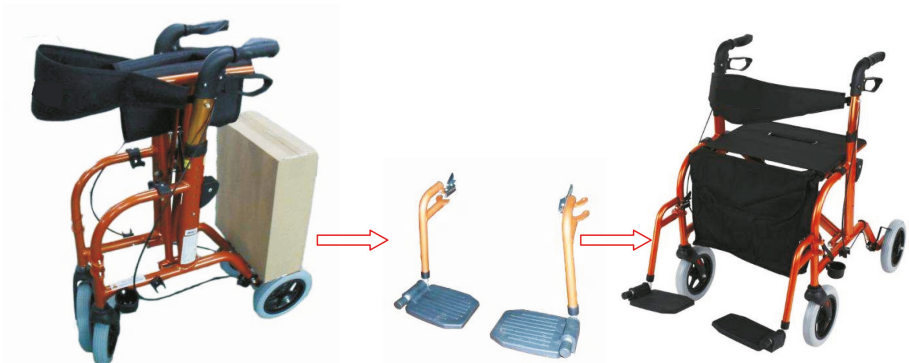
GB

1. Remove all packaging. The rollator will be folded as shown below. Firstly unfold the front castors and secure in place using the hand knobs as shown.
2. Fold out the rear leg as shown below and secure the legs in place with the hand knobs. Always take care to ensure that the hand knob fixed tightly.

Transport walking frame Campolina Article No. HCT-9103

1. Remove packaging. The rollator will be folded as shown below. Take the footrest out from the carton. Opening the rollator as below opening instruction.
2. Attach the footrest to the front frame.
3. Rollator assembled.

## HANDLE ADJUSTMENT

applicable to both models

The rollator's handles can be adjusted in height to suit different users. Simply depress the spring button and set to the desired height. Repeat this for the opposite side and make sure that both handles are secure and set at the same height before use.

## BRAKE OPERATION & ADJUSTMENT

applicable to both models

1. Pull up on brake handle lever to slow the rollator
2. Push down on brake handle lever to lock the rear wheels, pull up on the brake lever handle to release the lock

Brakes must be applied in the lock position when the seat is going to be used, pre check the lock by pushing forward the rollator.

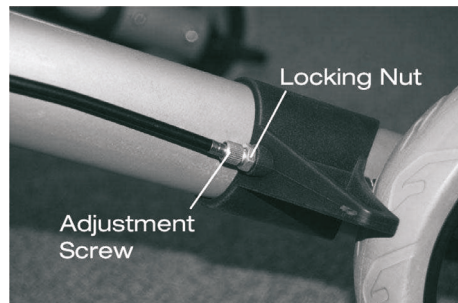
Brake adjustment is carried out like this:

Turn the adjuster screw anti-clockwise to increase the tension then lock the adjustment by turning the locking nut clockwise while holding the adjuster screw stationary.

Brake adjustment reduced to allow light pressure may be ineffective on the brake in the locked position.

GB

## MAINTANCE

**GB**

Over a period of time the brake cables may stretch, and if no further adjustments can be made the cables should be replaced. Seek the help of your dealer when replacing brake cables.

Castors should be checked that they are secure each time before using the rollator. Also check if the brakes operate correctly and if the handles and hand knobs are secure.

Should any part of the rollator appear loose or insecure stop using the rollator until it is inspected by an authorised dealer. The following shows maintenance schedule.

Weekly:

Check brake for malfunction. Check all wheels and castors for loose bolts, nuts and knobs.

Periodically:

Check hand grips for wear. Check if frame bolts are secure. Check if front and rear axle bolts are secure. Check tyres for wear or cracks.

Yearly:

Take your rollator to an authorised repairer for a full check, clean, service and lubrication of all moving parts one a year.

Serviceable components:

Wheels, hand grips, brake cables, brake lever, bearings, knobs, bag, bolt, nut

## CLEANING

Your rollator can be cleaned with a damp cloth or sponge and warm soapy water. Domestic anti-bacterial surface cleansers may be used if required.

Do not leave any cleaning residues on the hand grips or seat. They may be slippery when used. Do not use scouring pads or scrapers because this may damage surface coating.

Store your rollator in ventilated and dry place. Avoid oleic acid material to avoid corrosion.

## WARNINGS

1. Maximum working limit 130kg.
2. Do not stand on the rollator.
3. Ensure that both push handles are adjusted to the same height.
4. Seek advice from a trained health care professional or an approved freedom healthcare dealer for height adjustment of this product.
5. Always maintain a safe distance between your rollator and other pedestrians or objects.
6. Some components may cause harm or damage when they get into contact with other objects.
7. Do not attempt to replace the wheels or castors of your rollator. If they are faulty or require replacement please contact an authorised repairer.
8. Ensure that your rollator is opened fully (folding scissor arm is straight) before using it.
9. Ensure that your fingers are clear from the scissor arm when folding.

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KAPHINGST ONLINE AG
KAPHINGST ONLINE GmbH