# Aquatec® 90

- en** Raised toilet seat  
User Manual
- de** Toilettensitzerhöhung  
Gebrauchsanweisung
- fr** Rehausse WC  
Mode d'emploi
- it** Rialzo per WC  
Istruzioni per l'uso
- es** Elevadores de inodoro  
Manual de instrucciones
- nl** Toiletbrilverhoging  
Gebruiksaanwijzing
- da** Toiletforhøjelse  
Brugsanvisning
- fi** WC-istuimen korotus  
Käyttöohje
- no** Toalettseteforhøyelse  
Brukerveiledning
- sv** Toalettstolsförhöjare  
Bruksanvisning
- pt** Assento de sanita levantado  
Manual de instruções

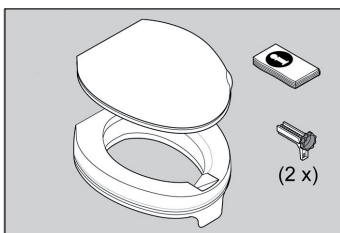
Fig. 1

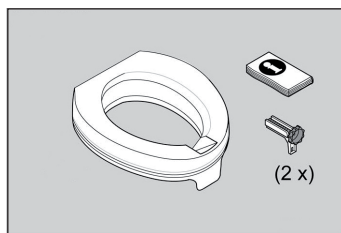
Fig. 2

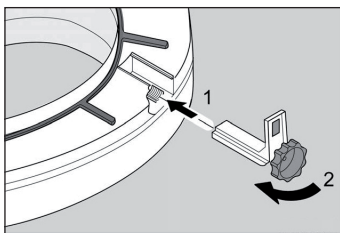
Fig. 3

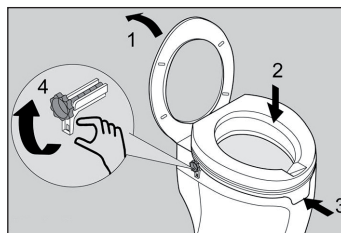
Fig. 4


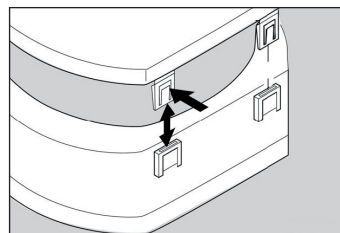
Fig. 5

This manual must be given to the user of the product. Before using the product, read this manual and save it for future reference!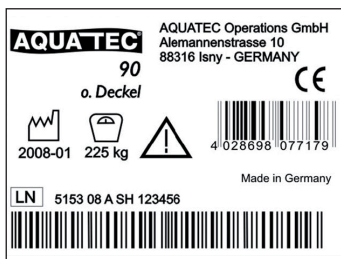

Yes, you can.

## Intended Use

en

The product serves exclusively to raise the toilet seat and is therefore only intended for installation on toilets. Any other use is prohibited.

### **WARNING** Risk of falling!

- Do not stand on this product.
- Do not use as a climbing aid.

Operating temperature: 10 - 40 °C

Max. user weight: 225 kg

## Service Life

The expected service life of this product is three years when used in accordance with safety instructions and intended use, stated in this manual. The effective service life can vary according to frequency and intensity of use.

## Safety

### **WARNING** Risk of injury!

- Do not use if defective.

## Scope of delivery

- Fig. 1, 2.

## Assembly

### **CAUTION** Risk of pinching!

- Take care during assembly.

- Fig. 3-5.

## Cleaning and disinfection

The product can be cleaned and disinfected using commercially available agents.

- Disinfect the product by wiping down all generally accessible surfaces with disinfectant (after disassembling the product, if possible).

All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean. For further information on decontamination in Healthcare Environments, please refer to 'The National Institute for Clinical Excellence' guidelines on Infection Control [www.nice.org.uk/CG002](http://www.nice.org.uk/CG002) and your local infection control policy.

- Clean by hand.
- Clean without scouring agent.
- Max. cleaning temperature 60 °C.

## Maintenance and periodic inspection

The product is maintenance-free, provided the cleaning and safety instructions outlined above are adhered to.

- Check product regularly for damage and to ensure that it is securely assembled.

## Re-Use

The product is suitable for repeated use. The number of times it can be used depends on how often and in which way the product is used. Before reuse, clean and disinfect the product thoroughly and check the safety functions, and reset if necessary.

## Disposal

The disposal and recycling of used devices and packaging must comply with the applicable legal regulations.

## Warranty

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business. Guarantee claims can only be made through the relevant specialist dealer.

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Algol-Trehab Oy
Aquatec Operations GmbH